






# ReadDesk Quick Start

The latest version of the full manual is available online at [www.issist.com](http://www.issist.com)

issist

## Reading

	KEYBOARD	MOUSE
Read	Spacebar	Click 
Delayed Read (15 seconds)	Hold Spacebar for 3 seconds	-
Menu	"Esc" Key	Click 
Pause	Spacebar	Click 
Play	Spacebar	Click 
Stop	"Esc" Key	Click 
Previous Word	Left Arrow Key	Click Last Word
Next Word	Right Arrow Key	Click Next Word
Previous Sentence	Up Arrow Key	Mouse Wheel Up

Next Sentence	Down Arrow Key	Mouse Wheel Down
Next Paragraph	Page Down Key	-
Previous Paragraph	Page Up Key	-
Go to Beginning	Home Key	-
Go to End	End Key	-

## VIEWS

	KEYBOARD	MOUSE
Change View	"V" Key	Click "Menu" ↓ Click "Reading" ↓ Click "View"

## ROTATE IMAGE

	KEYBOARD
Rotate Image	"R" Key

## TOGGLE COLOR

	KEYBOARD	MOUSE
Toggle Color	"C" Key	Click "Menu" ↓ Click "Color" ↓ Select new Color Scheme

SAVE/LOAD IMAGES SHORTCUTS for the  
KEYBOARD:

	KEYBOARD
Save	"S" Key

Load	“L” Key
Delete	“Delete” Key
Next Picture	“→”Key
Previous Picture	“←”Key
Export PDF	“P” Key
Export MP3	“M”Key

## AUTO SAVE IMAGES

	KEYBOARD
Start Auto Save	“S” Key for 3 seconds
Stop Auto Save	“Esc” Key

## AUTO READ IMAGES- IN LOAD MODE

	KEYBOARD
Start Auto Read	“Spacebar” Key for 3 seconds
Stop Auto Read	“Esc” Key

## SPELL A WORD

Press the CTRL key on any word.

## ZOOM IN/OUT

	KEYBOARD	MOUSE
Zoom In/Out	“+”/”-“ key	Right Click + Mouse Wheel Up / Down

If you have 2 Additional Mouse Buttons you can use the following controls to Zoom In/Out (See Figure 12).



MOUSE

Zoom In	Side Mouse Button 1
Zoom Out	Side Mouse Button 2

## MOVEMENT CONTROLS

Movement controls allow the user to move to different areas of the image.

	KEYBOARD	MOUSE
Up	8 (Num key)	Move mouse to Top.
Down	2 (Num key)	Move mouse to Bottom.
Left	4 (Num key)	Move mouse to Left.
Right	6 (Num key)	Move mouse to Right.

## MENU OPTIONS

	KEYBOARD	MOUSE
Enter	“Esc” Key	Click “Menu”

Menu		icon
Exit Menu	“Esc” Key	Click “Back” button
Up/Down	“Up”/”Down” Arrow Keys	Move mouse to Top/Bottom
Select Option	“Enter” Key	Click on Option
Enter Submenu	“Enter” Key	Click on Submenu
Previous Menu	“Backspace” Key	Click on “Back” button



Issist Assistive Technologies Inc.

[www.issist.com](http://www.issist.com)